STERILE ISOTONIC SOLUTION (0.9% NaCl)
Syringe pre-filled with sterile isotonic solution intended for the flushing of vascular access devices
Sterile - Latex-free - Disposable

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QUALITATIVE AND QUANTITATIVE COMPOSITION
SODIO CLORURO 0.9%
Eccipient: WATER FOR INJECTIONS q.s. to 100 mL.
The solution of Sodium Chloride 0.9% is a sterile isotonic aqueous sterile that presents approximately the same osmolality of the fluids extracellulare. Non è irritante per i tessuti.

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AVAILABLE VARIANTS

STERILIZED USING MOST HEAT	STERILIZED BY GAMMA RADIATION
5 mL pre-filled disposable syringe, filled to 3 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 3 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 3 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.	5 mL pre-filled disposable syringe, filled to 3 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 3 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.
5 mL pre-filled disposable syringe, filled to 5 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 5 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.	5 mL pre-filled disposable syringe, filled to 5 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 5 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.
5 mL pre-filled disposable syringe, filled to 10 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 10 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.	5 mL pre-filled disposable syringe, filled to 10 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 10 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.
5 mL pre-filled disposable syringe, filled to 20 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 20 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.	5 mL pre-filled disposable syringe, filled to 20 mL, with Luer Lock fitting, individually packaged in a paper-PET/PP pouch. 5 mL pre-filled disposable syringe, filled to 20 mL, with Luer Lock fitting, individually packaged in an aluminum-PET/PE pouch.

The volume indicated on the label refers to the syringe filling volume. The graduated scale on the label of the immediate packaging (syringe) has no measuring function.

INTENDED USE AND CLINICAL BENEFIT
This Medical Device is intended for use exclusively in all clinical procedures where flushing of vascular access devices is required. An effective flushing of vascular access devices helps to prevent occlusion and minimizes the risk of infection/contamination of the vascular access devices; it also helps prevent the accumulation of blood and/or medicinal products; reduces that may precipitate in the catheter lumen. Additionally, flushing can prevent foam formation.

INDICATIONS FOR USE
This Medical Device is indicated for:
- maintaining the patency of vascular accesses;
- preventing Venous Accesses (CVA);
- post-flushing;
- delays attachment and detachment procedures;
- preventing management of vascular accesses;
- procedures need pre- and post-administration of medicinal products or transfusions to the patient.

PRECAUTIONS
Do not administer via intravenous route directly.
Do not use the product for purposes other than its intended use.
Do not use the product if the packaging is open or damaged, if the syringe is not fully intact or if the cap is damaged in such a way as to cause leakage from the syringe.
Do not use the product if the solution is not colourless or contains obvious particulate matter.
Once opened, the device must be used immediately and disposed of after use.
Single use only. Dispose of any partially used product, the reuse of which could cause cross contamination between patient and operator.
Do not resterilize. Resterilization carries a risk of cross infection for the patient and/or user.
Do not draw air into the syringe and do not allow air to be trapped in the fluid path.
Carefully avoid contact between the flushing solution and incompatible products.
If used in a home environment, keep the product out of the reach of small children.

OVERDOSE
Not applicable if properly used in relation to intended use and in relation to catheter volume.
Do not use to reconstitute lyophilized products or dilute medicinal products.
Side effects described in the literature that may occur in relation to vascular access management include device occlusion, injection site infections or infections associated with blood reflux, thrombosis or phlebitis. Hypertensive and water retention, in cases where the administration of sodium or chloride could be clinically harmful.
If an adverse reaction occurs, discontinue the infusion, assess the patient. Appropriate therapeutic countermeasures and save the residual fluid for examination, if deemed necessary.

INTERACTIONS WITH OTHER SUBSTANCES OR MEDICAL PRODUCTS
None, in relation to the intended use of the product, i.e. vascular accesses flushing.

INSTRUCTIONS FOR USE
Disposable device intended for a single patient.
Ensure strict asepsis throughout the procedure, the healthcare professional must comply with the procedures of their institution, adopting an aseptic technique for the entire procedure.
Check the validity of the expiry date indicated on the sealing side of the pouch.
Open the pouch and take the syringe out. Verify the correct positioning of the syringe cap, the transparency of the solution and the absence of particulate matter. If necessary, place the syringe on a sterile field.
Before removing the closure cap, apply pressure to the plunger to release the pressure plug. This ensures a smooth and uniform emptying of the syringe.
Remove the closure cap, ensuring that no contact contamination of the patient's Luer cone occurs.
Consult the instructions for use.
After use, do not use unused volumes of solution, but discard them.

PERIOD OF VALIDITY
3 years in unopened packaging.
Storage
Store the syringes tightly closed, away from direct light and heat sources, at a temperature between +5 °C and +30 °C. Do not freeze. The expiry date refers to the product in unopened packaging, correctly stored. Do not use the Medical Device after this date.

REPORTING OF SERIOUS INCIDENTS
The user and/or patient is advised to report any serious incidents occurring in relation to the Medical Device to the manufacturer and the competent authority of the Member State where the user and/or patient is established.

METHOD OF DISPOSAL
The Medical Device is of disposable type. After use, dispose of the product in the appropriate hospital containers and according to the procedure of the health facility where the product is used or according to the regulations of your municipality. Do not dispose in the environment.

SYMBOLS USED ON THE LABEL	
	Manufacturer
	STERILE 1 Sterilized using moist heat
	The exterior of the outer packaging (paper-PET/PP pouch) is a sterile barrier system
	Single sterile barrier system
	Single sterile barrier system with protective packaging outside
	Do not resterilize
	Temperature limits for storage
	Keep away from light and heat sources
	Sterilized by gamma radiation
Medical Device compliant with EU Regulation 2017/745	
D.B.M. Single-member limited liability company	
Via dell'Industria, 2 - 23030 Tovo di Sant'Agata (SO) - Italy T +39 0342 77615 - info.dn@numtec.eu - www.dombitalia.it	

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The volume reported in the label refers to the volume of the syringe. The scale graduated, reported at the top of the syringe, is for informational purposes only and does not have a measuring function.

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SVORADSSAGGIO
Not applicable in relation to the destination of use and to the volume of the product.

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